FR

PRESENTATION

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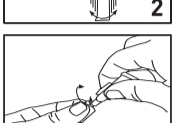
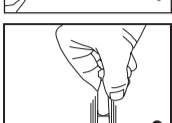
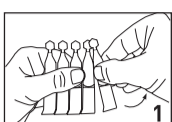
EN

PRESENTATION

NEOVIS® TOTAL is an ocular lubricant in a sterile hypotonic emulsion form of 0.18% sodium hyaluronate (obtained by fermentation), presented in preservative-free single-dose units of 0.4 ml.

COMPOSITION

Sodium hyaluronate (0.18%), triglycerides and phospholipids, lipoic acid, hydroxypropylmethylcellulose, sodium citrate, sodium hydroxide, citric acid, sodium chloride and water for injection. NEOVIS® TOTAL is an emulsion which stabilizes the tear film. NEOVIS® TOTAL contains sodium hyaluronate, a naturally-occurring component of the eye that helps to protect and hydrate the ocular surface. Due to the physical properties of sodium hyaluronate, NEOVIS® TOTAL has a high capacity to retain water and is viscoelastic, enabling the formation of a homogeneous, lubricating, protective film on the surface of the eye. In this manner, it acts on the aqueous layer to hydrate and protect the ocular surface. NEOVIS® TOTAL is hypo-osmolar to compensate for harmful effects (inflammations) caused by tear hyperosmolarity in cases of dry eye. Sodium hyaluronate also possesses mucomimetic properties enabling NEOVIS® TOTAL to anchor the tear film. To maximize local tolerance, it does not contain preservatives or surfactants.

IN WHICH CIRCUMSTANCES SHOULD NEOVIS® TOTAL BE USED?

NEOVIS® TOTAL is recommended in cases of moderate to severe sensations of dry eyes.

HOW SHOULD NEOVIS® TOTAL BE USED?

NEOVIS® TOTAL is used in the eye. Carefully wash your hands before instillation.

- Open the sachet and remove a single-dose unit from the strip, making sure to store the remaining single-dose units in the original packaging.
- Always shake the single-dose unit before instillation. Break the seal. The single-dose unit is open.
- Hold the single-dose unit vertically (cap on top) and turn the flat part completely to break the seal. The single-dose unit is open.
- Tilt head back and position the tip of the single-dose unit above the eye. Gently pull the lower eyelid down with the index finger while looking upward, and gently press the body of the single-dose unit to release a drop of NEOVIS® TOTAL.

Unless otherwise recommended, instil 2 drops as often as needed. After application, spontaneous blinking will distribute the emulsion over the eye's surface to form a transparent and durable film. NEOVIS® TOTAL can be used when wearing contact lenses.

PRECAUTIONS FOR USE

For external use only. Do not ingest. Store between 2 and 25 °C.

- Carefully wash your hands before instillation.
- Always shake the single-dose unit before instillation.
- Do not touch the tip of the single-dose unit with hands, eyes or eyelids (risk of microbial contamination).
- These drops are presented in a single-use container: the single-dose unit should be discarded immediately after use and should not be stored for a subsequent instillation (risk of microbial contamination and loss of product efficacy).
- Do not use if the single-dose unit is damaged (risk of microbial contamination).
- Do not use after the expiry date shown on the single-dose unit and the packaging (risk of microbial contamination). If your doctor has prescribed another eye drop, allow 15 minutes between applications of the two eye drops (risk of change in the effectiveness of the eye drops). If symptoms persist during the use of NEOVIS® TOTAL, consult your doctor. Keep out of the reach and sight of children.

CONTRAINDICATIONS

Do not use in case of hypersensitivity to one of the components.

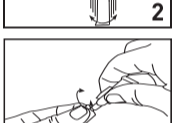
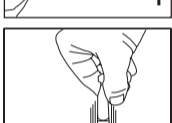
ADVERSE EFFECTS

In very rare cases, temporary symptoms such as mild conjunctival irritation, the sensation of having a foreign object in the eye, redness, burning, and blurry vision may occur. If an adverse effect persists, or if there is an adverse effect not described in this leaflet, stop using the product and consult your doctor.

Medical device compliant with the requirements of the amended directive 93/42/CEE.

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