TEOSYAL® PURESENSE 1
 Le Chevallier, Rue de la Loi, 103, 1201 GENEVE
 Tel. +41 22 342 22 22 Fax +41 22 342 22 23
 info@teosyal.com

EN Instructions for use

Description

TEOSYAL® PureSense Kiss® is a sterile, transparent, colorless, hyaluronic acid of non animal origin containing 0.3 % lidocaine hydrochloride by mass for its anesthetic properties. Each box contains two syringes prefilled with TEOSYAL® PureSense Kiss®. For each syringe, the box contains two sterile 27G1/2 needles and two syringeability labels (to be given to the patient, and one to be kept by the doctor in the patient's file). The volume of each syringe is shown on the box as well as the correct filling.

Composition
 Cross-linked hyaluronic acid 25 mg
 Lidocaine hydrochloride 0.3 mg
 Phosphate buffer pH 7.3 q.s. ad 1 ml

Therapeutic indications for TEOSYAL® products

Reconstruction, surgery, filling of depressions due to scars, reconstruction of volumes lost by lipotrophy in the face, reconstruction of volumes, correction of wrinkles, aesthetic indications for TEOSYAL® products

Modification of the anatomy of aged skin: restoration of volume, filling of wrinkles, depression and wrinkles, restoration of skin hydration.

TEOSYAL® PureSense Kiss® is indicated to fill the wrinkles and increase the volume of the face. It provides correct medium to deep wrinkles in the corners of the lips. Lidocaine is a local anesthetic designed to reduce the sensation of pain.

Mode of action

TEOSYAL® PureSense Kiss® is injected into mucosa for the correction of the lips and into the deep dermis for the treatment of cutaneous depression. TEOSYAL® PureSense Kiss® treats volume loss, corrects wrinkles and restores lip volume. TEOSYAL® PureSense Kiss® is biodegradable and is slowly resorbed over several months. The duration of the effect is variable, depending on the depth of the wrinkles and/or volume of the lips in need of treatment, to the volume of filling, the depression and wrinkles, restoration of skin hydration.

TEOSYAL® PureSense Kiss® is indicated to fill the wrinkles and increase the volume of the face. It provides correct medium to deep wrinkles in the corners of the lips. Lidocaine is a local anesthetic designed to reduce the sensation of pain.

Mod of action

TEOSYAL® PureSense Kiss® is injected into mucosa for the correction of the lips and into the deep dermis for the treatment of cutaneous depression. TEOSYAL® PureSense Kiss® treats volume loss, corrects wrinkles and restores lip volume. TEOSYAL® PureSense Kiss® is biodegradable and is slowly resorbed over several months. The duration of the effect is variable, depending on the depth of the wrinkles and/or volume of the lips in need of treatment, to the volume of filling, the depression and wrinkles, restoration of skin hydration.

Contra-indications

TEOSYAL® PureSense Kiss® must not be used:
 -for the correction of wrinkles in the periorbital area;
 -for injections, other than intradermal,
 -do not inject into the eyelids,
 -in the case of patients with allergic disorders, inflammation or infection at the treatment site or near to this site,
 -in the case of patients having a known hypersensitivity to hyaluronic acid or in combination with peeling, laser or ultrasound-based treatments,
 -in the case of patients with autoimmune diseases,
 -in the case of patients with cardiac diseases and/or undergoing treatment for heart diseases (beta blockers),
 -in the case of patients with hepatocellular insufficiency and/or undergoing treatment for liver disease,
 -in the case of patients suffering from epilepsy or porphyria,
 -it is inadvisable to inject TEOSYAL® PureSense Kiss® into sites in the presence of other filling implants which have not been researched,
 -in pregnancy, breastfeeding mothers, or in children,
 -in the case of patients with a history of infection, this product contains an active component which may cause a positive result in a drug test. Do not inject into the eye.

Dosage and method of administration

The injection of TEOSYAL® PureSense Kiss® is reserved for practitioners trained in the injection techniques and products mentioned in this manual, and injects patients must be questioned regarding their past medical history and informed regarding the possible outcomes of treatment and of potential undesirable effects. Correction sites must be properly disinfected. It is advised to perform injections using the correct technique. TEOSYAL® PureSense Kiss® should be injected slowly into the deep dermis to correct wrinkles and into the mucosa to restore lip volume. The linear threading injection technique. If the injection is made too deeply, it will indicate that the injection will be too superficial, which it is advisable to avoid. The result of the injection will be irregular. The duration of the effect depends upon the correction required. It is however recommended not to exceed 2 mL per session. It is important not to overcorrect. The distribution of the product is carefully to ensure a uniform distribution of the product at the corrected sites.

Side effects

The practitioner must inform the patient that there are potential side effects related to the implantation of the product. The possible side effects are listed below. No pain is felt in the various vascular territories.

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Mounting the needle on the syringe

For optimum use of TEOSYAL® PureSense products, it is important that the needle is mounted on the syringe following the 4 steps detailed in the diagrams (see figures 1 to 5). If there is a sensation of pressure or pain during the injection, halt the process and change needles.

Precautions for use

The practitioner must consider the fact that this filling product increases the risk of infection. Check the use-by date and the integrity of the packaging. Do not use the product if the seal of the packaging is damaged. The patient should be advised not to take high-dose vitamin E, aspirin, anti-inflammatory drugs or other anticoagulants before the injection session. Advise the patient not to use make-up during the 12 hours following injection. Advise the patient to avoid saunas, hammams, etc.) during the week following treatment. At the end of the treatment session, it is recommended to use a cooling gel. This device is intended for single use only. Sterility is not guaranteed in the event of re-use. The gel can be used to soothe the skin after the injection, but not hindering its extrusion through a needle.

Storage conditions

Store between 2°C and 25°C, away from direct sunlight. Do not use if there are no visible signs of damage to the packaging before use.

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