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This little leaflet is full of important information - please keep it & read it before using your **Flo** tampon!

ABSORBENCIES:

• Lite, less than 6 grams
• Regular, 6-9 grams
• Super, 9-12 grams
• Super Plus, 12-15 grams

Always use the lowest absorbency for your flow.

BE IN THE Flo

- Only use tampons during menstruation.
- You can wear a tampon at night if you sleep for less than 8 hours - otherwise use a pad.
- Don't flush tampons, applicators or wrappers - they will end up in the ocean!
- When inserting a tampon, some positions may be more comfortable:
 - sitting on the toilet with your knees wide apart
 - standing with your knees bent and legs apart
 - standing with one leg on the toilet
- Use your thumb and middle finger to grasp the base of the outer applicator tube when inserting.
- Your vagina slopes towards the bottom of your back, not straight up. Insert the tampon at an upward angle & remove by pulling the string at a downward angle
- **KEEP CALM & Flo ON - IT WILL ALL BE OVER SOON!**

Flo

**WE'RE HERE FOR YOU!
SAY HIYA@HEREWFLO.CO
OR HOP OVER TO
HTTP://HEREWFLO.CO
TO LEARN MORE. WE'RE ALSO
SOCIAL-MEDIA-ISING
AS @HEREWFLO.**